NORMBAU

PASSION FOR CARE

Cavere FlexCare

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Load

Max. permissible user weight of all listed products: 4kg

7100 040 7100 041
7100 050 7100 051

Max. permissible user weight of all listed products: 10kg

7100 010
7100 011
7100 012

Warning notices

Installation is to be carried out by a specialist and only with NORMBAU mounting sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material.

Observe the information on the max. permissible user weight and the instructions for mounting!

Care and cleaning instructions

Powder-coated products from NORMBAU have a smooth, easy-to-clean surface. In order to maintain the decorative design and color of the coating of our products for years, cleaning and care must be carried out professionally. Slightly adhering dirt can be removed with a sponge and water to which a neutral cleaning agent (e.g. dishwashing liquid) is added. Immediately after each cleaning process, rinse with pure, cold water. A chamois or microfiber cloth removes the remaining wiping marks or drops of water. Abrasive or strong acidic/alkaline cleaning agents should never be used. With metallic coatings, heavy soiling should be cleaned as soon as possible, since dried and old dirt can often only be removed abrasively and with the risk of damage to the powder coating or a change in color.

Common wiping disinfectants can be used.

Intended use

NORMBAU products are designed for use in wet rooms, at home, in care facilities and hospitals. Use by children is not permitted.

Articles that are no longer required must be disposed of in the prescribed manner. Listed articles are made of recyclable polyamide and metal components.

Maintenance

NORMBAU products are maintenance-free.

Attention

There is no toxic hazard from the material used by NORMBAU. Tested according DIN EN ISO 10993-5:2009. Sun exposure or cleaning with hot water can reach temperatures higher than 41°C, which can lead to skin damage / burns.

Caution, risk of injury

Disclaimer of liability

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions. According to the current regulation, the warranty is two years.

Under the intended conditions of use and in compliance with the guidelines for maintenance and cleaning, the intended lifespan of the article is five years.

If you would like more information about the products, visit us at www.normbau-extranet.de or contact us via customer service +49 (0) 7843 704 0.

For an enlarged view, you can find these assembly instructions by entering the article number on the homepage

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NORMBAU GmbH